CP-ONE PLUS






Thinking of All







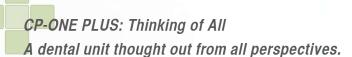

The dental chair and unit are the key factors in ensuring safe care with accurate diagnosis, precise treatment, and reliable counseling.

Thinkin g of Communication

Thinkin g of Patient Comfort

Thinkin g of Operator Comfort

The CP-ONE PLUS was designed by incorporating doctors' requirements and desires one by one, from the treatment space all the way down to minute details that will be recognized through doctors' fingertips.

An ideal treatment environment, the CP-ONE PLUS is the **Thinking of All** dental chair and unit, the answer to doctors' aspirations, made possible only through the fusion of the expertise and technological leadership of Takara Belmont.



The CP-ONE PLUS is the successor to the CP-ONE, or Communication Product No. 1.

What is communication through the CP-ONE PLUS?

Relaxing the patient is the first step towards providing efficient treatment and facilitating informed consent.


The CP-ONE concept allows for communication in any position. Patients are able to communicate their needs and level of comfort, and doctors are able to communicate their intentions and explanations.


The CP-ONE PLUS succeeds in taking the concept of the CP-ONE and improving it with the latest functionality.



Monitor bracket (optional)

A monitor mounted on the doctor table will encourage more effective communication, especially in terms of informed consent.

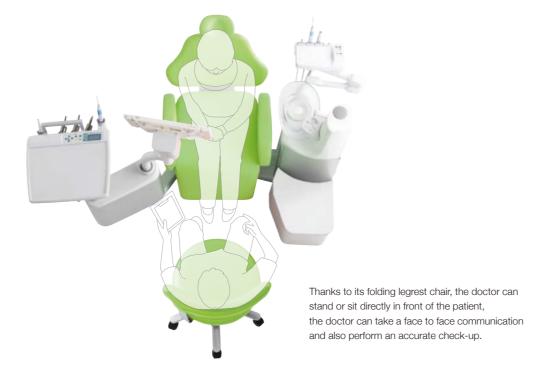

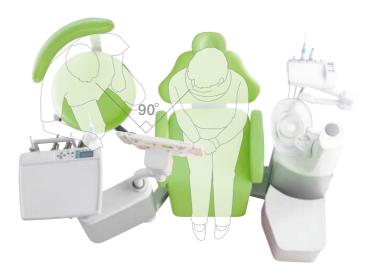

The CP-ONE PLUS, like the CP-ONE, provides a 90-degree eye-contact position that allows the viewpoints of both the patient and the doctor to converge. The 90-degree eye-contact position is conducive to a natural, stress-free atmosphere for discussion and provides for relaxed, thorough communication.




The dental chair is the medium through which the patient entrusts him/herself. The CP-ONE PLUS chair has been ergonomically and luxuriously designed to provide true comfort for all patients in addition to reliability.

The CP-ONE PLUS is not only a dental chair and unit, but a communication tool that builds trust.

Folding legrest chair with a low initial height of 400 mm

Sitting down and getting up is easier for children, the elderly, and those with limited mobility, as the chair can be accessed either from the side or directly from the front. A legrest heater is available as an option.

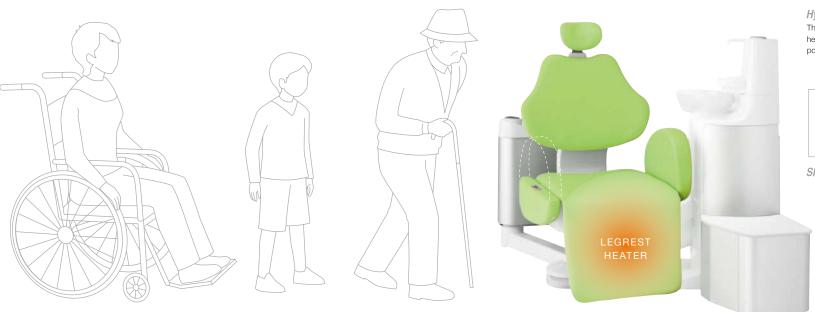



New shockless system

The new shockless system (slow slide valve) eliminates any jarring movements of the chair, keeping the patient calm and comfortable for a stress-free treatment.








Hydraulic-powered headrest

The axis headrest is designed based on ergonomic analysis of the head and its movement. The doctor can adjust the headrest position from the table or stick switch.



Slim type armrest (optional)

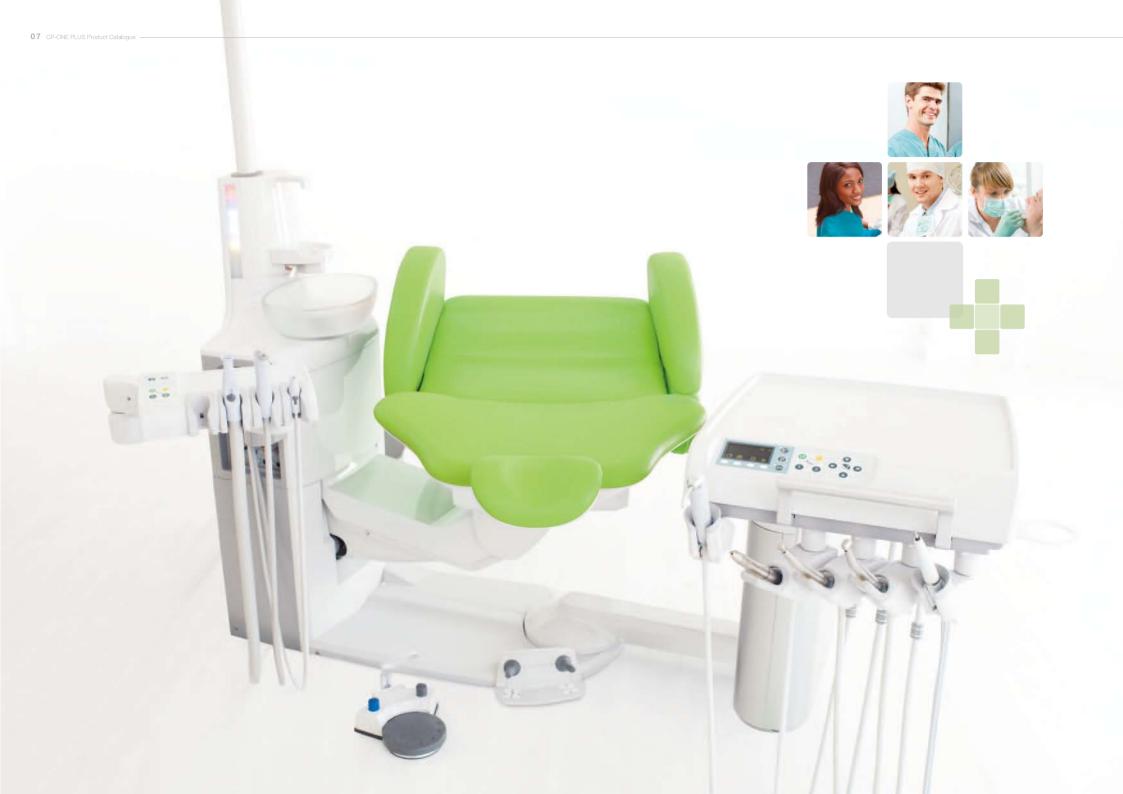

Thinking of Operator Comfort

The CP-ONE PLUS not only addresses operators' daily requirements, but also meets unfulfilled demands. The doctor table provides an extra smooth transfer of instruments into and from the holder, a high level of visibility with the organic EL display, and a simple interface for various controls that encourage the doctor to concentrates his/her treatment without stress.

Wide range of positioning

Inherited from the CP-ONE is the arc delivery system; the doctor table can be smoothly moved for the 6 to 2 oíclock position. Furthermore, with the expanded assistantís area, it is compatible with 2-hand, 4-hand, and 6-hand operations.











Newly developed foot controller

It is controlled by pressing and/or turning the disk, which provides precise handpiece control.



New stick switch for chair and/or headrest control

Easy to operate chair (manual and/or preset position) and/or headrest controls keep hands sterile. One is standard, and an extra is optional to provide further functionality.


Doctor Table



New control panel

Versatile function with a simple, intuitive control panel and highly viewable organic EL display

SIMPLE INTERFACE

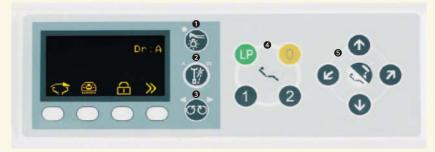

Membrane Switch

- 1 Handpiece light On/Off
- 2 Handpiece air/water control
- 3 Micromotor direction control
- 4 Chair control (preset position)
- **5** Chair and headrest manual control

DIGITAL INDICATOR



Hi-speed turbine control

4 types of power modes are available.

 $(LL\rightarrow LOW\rightarrow STD\rightarrow HI)$



Micromotor control

Highly configurable power control between 100ñ40,000 rpm.



Ultrasonic scaler control


Power control by 10 steps (1→10) for Scaling/Perio/Endo mode respectively

Cuspidor Bowl and Assistant Holder




New assistant table with intuitive control panel

Handpiece holder is detachable and autoclavable.

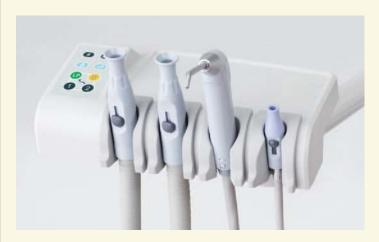



- 1 Chair preset switch
- 2 Chair auto return switch
- 3 Chair last position switch
- 4 Cup filler switch
- 6 Bowl flush switch
- 6 Dental light switch
- Extra switch

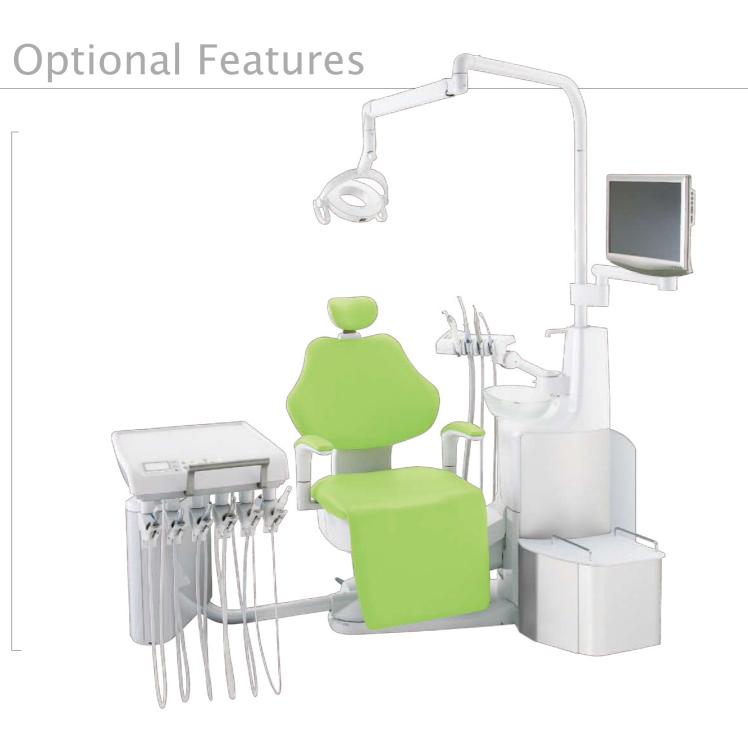

Articulate balance arm for assistant holder



Longer arm for doctor table



Large cuspidor unit cover for amalgam separator



Dental Light

Monitor bracket on light post



Amenity box on junction



Clean water system



Monitor bracket



Slim type armrest





Basic Specifications

- Focal distance: 650 mm
- Color temperature: 5,000 Kelvin at any intensity
- Color rendering index: more than 90
- Light intensity: 4,000–32,000 lux stepless control
- Light intensity composite mode: 5,500 lux
- LED modules: 10
- Power requirement: 100-240 VAC
- Light pattern: 85 x 155 mm





io5000TA triple-axis dental light





Lineup



CP-ONE PLUS DIMENSIONS



