## **EURUS Maintenance, Cleaning, and Disinfection**


## After each patient Daily 1)Disassembling and washing the parts 1) Disinfection of Dental unit, chair, and light (except light lens). \*Holder marked with " \* " sleeves 1. Moisten a soft cloth with FD366 manufactured by Durr. 2. Wipe the product with the cloth. \*Dirt filter \*Drain trap **CAUTION** Do not spray directly! Cuspidor bowl Durr FD366 Reattachement 3. Choose a dry soft cloth and wipe the To reattach the drain trap, align Line with product to dry. $Mark \nabla$ . Vacuum handpiece & Saliva ejector handpiece 2) Flushing handpieces & cupfiller. 1. Press the function switch. A liner 1. Wash the surface. 2. Insert the Handpieces. 2. Wash the interior and sliding part. 3. Press "Flushing." 3. Wash the inaccessible parts with brush. 3) Cleaning Vacuum/ Saliva ejector line. Suction a cup of water (100ml or more). 4. Autoclave 4) Cleaning Solid collector. 2) Cleaning Vacuum line/Saliva ejector line. Wash the components under running Use Orotol plus manufactured by Durr water. **A** CAUTION Reattachement Do not use any other liquid chemical. Align Groove with Claw. Correct position: The wider part of the knob is located in the upper section.

# **EURUS Maintenance, Cleaning, and Disinfection**

#### Weekly



- 1) Cleaning for Dental Unit, Chair, and light (except Light lens).
- 1. Wipe with a dry soft cloth.

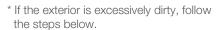



1. Moisten a soft cloth with water containing approx. 10% of netural detergent and wipe the product with the moisten cloth.

#### Weekly



4. Lightly wipe the light lens with a soft cloth or remove any dust from the surface with an air duster.

#### Dry soft cloth



Do not apply FD366 to the light lens.



Air duster



**A** CAUTION Do not spray directly!



2. Moisten a soft cloth with water and wipe the product.



Reattachment the lens cover.

5. Insert the claws of the lens cover.





6. Slide the lock lever to the right and hold it.



3. Wipe the product with a dry soft cloth.



7. Apply the cover in contact with the light head and retrun the lever.





### Monthly



Sponge

1) Discarding the oil in the Oil mist

When oil reaches to the line on the oil reservoir, discard it.



2) Cleaning Light lens Detache the lens cover. 1. Slide the lock lever to the right.



If the hygroscopic sponge (consumable) is excessively dirty or has excessive oil, replace it.



- 2. Incline the lens cover.
- 3. Pull it up to detach it.